

Eversense Office Checklist

Patient Education Prior to Procedure Day

- ☐ Set up appointment with patient and instruct patient to wear a loose, short-sleeved or sleeveless shirt for placement in arm
- ☐ Encourage **ALL** first-time users to attend Eversense Product Training provided by the Clinical Education Team. The training covers pre- and post-insertion.
- ☐ Review the Eversense product with patient:
 - System Components
 - Sensor, Transmitter, Mobile App, Adhesives
 - Download Eversense 365 App to compatible mobile device and set up account:
eversensecg.com/compatibility
 - Therapy Considerations
 - Contraindications
 - Fingerstick requirements

Inventory Management

- ☐ Open Eversense shipment immediately when it arrives
 - Remove Eversense 365 Sensor Kit
 - Label sensor package with patient name (if applicable)
 - **Store sensor package in refrigerator until procedure day**
 - Bundle Insertion Tools Kit, Transmitter Kit, and Adhesives
 - Label bundle with patient name (if applicable)
 - Ensure procedure (Vygon) kit is available
- ☐ **Items Not Included:** The following items are to be provided by clinic and ordered to allow for delivery before procedure day
 - ___ Local anesthetic for injection, e.g., lidocaine 1% or 2% with or without epinephrine (~2 mL per procedure)
 - ___ Non-latex sterile surgical gloves, healthcare provider-preferred size
- ☐ Confirm expiration dates on all products and supplies

Procedure Day Basics

- ☐ Set up procedure room
 - ___ Ensure space for sterile field setup (e.g., Mayo stand), and make sure countertops and exam table are clear and disinfected
 - ___ Gather supplies and check integrity and expiration/use-by dates
 - ___ Confirm correct sensor and serial number
 - ___ Connect transmitter to charger and charge for approximately 15 minutes
- ☐ Confirm allergies and review/sign informed consent
 - ___ Confirm no allergy to dexamethasone or dexamethasone acetate
 - ___ Confirm no allergies to other supplies used for procedure (e.g., lidocaine or iodine) or other Eversense product-related materials
 - ___ Confirm correct procedure and location
 - ___ Review procedure expectations and risks, and sign informed consent form with patient
- ☐ Post-procedure
 - ___ Review and provide incision care instructions (Eversense Next Steps document in Smart Transmitter Kit)
 - ___ Enter **System Setup** in Eversense 365 App and follow steps to pair the smart transmitter, link the sensor, and begin the 24-hour warm-up phase
 - ___ Send patient home with Transmitter Kit (including charger and User Guide), and Adhesive Patch Kit
 - ___ Implement follow-up plan/next office appointment (if applicable)
- ☐ Encourage ALL first-time users to attend post-insertion Eversense Product Training provided by Clinical Education Team

Supply List

Supply Item	Eversense 365	Description *Check Expiration/Use by dates on all Products
Sensor		Sensor Kit* - Refrigerate immediately upon arrival - Confirm correct sensor type for patient (DME/RX or Bundled Payment) - Label sensor with patient name - Back-up sensor recommended for office
Insertion Tools		Insertion Tool Kit* - Includes incision template 1 kit ships automatically with every sensor ordered - Back-up tool kit recommended for office
Customer Procedure Kit		Procedure (Vygon) Kit* 1 kit ships automatically with every sensor ordered - Removal procedure only: Procedure (Vygon) kit should be ordered from Gemco. - Back-up procedure (Vygon) kits must be ordered separately. Ask your Eversense representative for assistance.
Adhesives		Adhesives Patch Kit* 300 White & 90 Clear
Transmitter Kit		Smart Transmitter Kit* Kit includes: charging components, transmitter and User Guide

For additional questions/concerns:

Contact Customer Service at:

Phone: 844-SENSE4U (736-7348) or Email: support@eversenseddiabetes.com



Patients can sign up for Eversense Product Training today!

Our clinical education team will guide them through everything they need to know to confidently prepare for their Eversense sensor placement.

◀ Scan here or visit bit.ly/eversense-virtual-classes to view upcoming dates and secure a spot!



© 2026 Senseonics, Incorporated.
All rights reserved.

Manufactured by Senseonics, Inc.
20451 Seneca Meadows Parkway
Germantown, MD 20876-7005 USA
eversensecg.com

The Eversense® 365 Continuous Glucose Monitoring (CGM) System is indicated for continually measuring glucose levels for up to one year in people (18 years and older) with diabetes. The system is indicated for use to replace fingerstick blood glucose (BG) measurements for diabetes treatment decisions. Fingerstick BG measurements are required for calibration one time a week after day 13, and when symptoms do not match CGM information or when taking medications of the tetracycline class. The sensor insertion and removal procedures are performed by a health care provider. The Eversense 365 CGM System is a prescription device; patients should talk to their health care provider to learn more.

For safety information, see bit.ly/eversensesafety

Eversense, Eversense 365 Continuous Glucose Monitoring, and the Eversense logo are trademarks of Senseonics, Incorporated. Other brands and their products are trademarks or registered trademarks of their respective holders and are used solely for informative purposes. No relationship or endorsement should be inferred or implied. MKT-001756 Rev1