

Chart Note Example for Insertion and Removal

Informed consent obtained after discussing procedure and possible complications. Allergies documented regarding procedural products.

Insertion:

Insertion area located in right/left upper arm and marked. Area cleaned, draped, and local anesthesia injected. Incision made and sensor deployed.

Pressure applied to incision site, closed with steri-strips or another method, and dressed. Patient tolerated procedure well. No complications noted. Incision care instructions provided. Patient instructed to call if signs/symptoms of infection develop.

Sensor linked to transmitter and warm-up phase initiated. Signal strength documented as excellent/good/low.

Instructions were given for activating the system and the patient was given training materials on system usage.

Removal:

Previous sensor located by palpation and marked. Skin prep applied, draped, and local anesthesia injected.

Incision made. Sensor located and removed successfully.

Pressure applied to incision site, closed with steri-strips or another method, and dressed. Patient tolerated procedure well. No complications noted. Incision care instructions provided. Patient instructed to call if signs/symptoms of infection develop.

This document is intended for trained providers as an example of common chart notes used to document events of the Eversense E3 CGM Insertion and/or Removal procedure. It is not intended to be used as-is and should be amended to reflect each individual procedure and to meet individual standards of practice.